

ASSEMBLY INSTRUCTION FOR FACE SHIELD

Step 01 Remove the protective film from the transparent sheeting.


Note: There may be nearly invisible film on both sides.



Step 02 Hold the sheeting at a 90 degree angle and attach it to the left side of the visor.




Step 03 Rotate the sheeting.

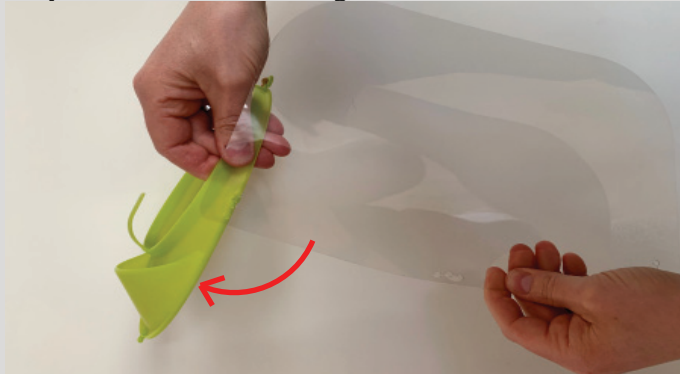


Step 04 Attach the other side of the sheeting to the visor. To adjust the angle of the sheeting, use the top, middle or bottom hole on the left and right side.



Step 05 Mount the center of the sheeting to the center of the visor.

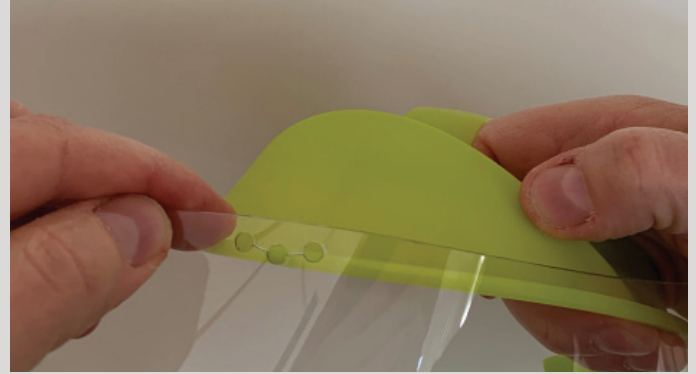

Step 06 Attach the rubber band to the back.

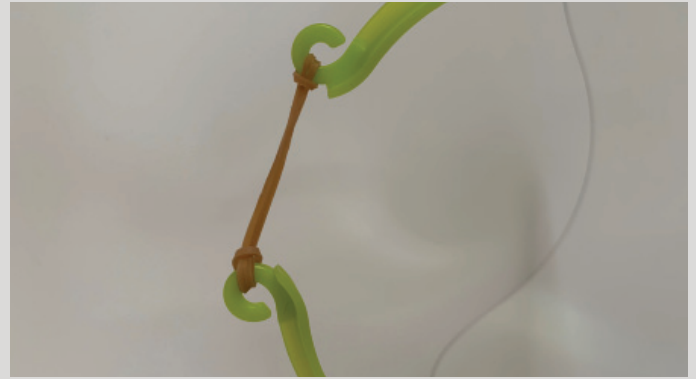

Step 07 The face shield is ready for use.



Technical information

- Transparent sheeting:
 - Clean with soapy water and a microfiber cloth.
 - Handles up to 60 degrees of heat.
 - Prolonged use of alcohol-based cleaning products will tint the sheet.
- Dimensions: H: 250mm, W: 164mm, D: 230mm
- Weight: 75 grams / 2.6 oz

This manual should be available to each user.

INSTRUCTIONS FOR USE – IFU

This manual should be available to each user

Storage

Store the products in cardboard boxes until required. When stored in the packaging, the product will keep for 36 months.

The product should not be stored in direct sunlight or in excessively hot conditions.

Intended use

The product is intended to protect against droplet contamination.

Everyday use

Visor cleaning is best done with a microfibre cloth, water and soap. Soap with abrasives will scratch the visor. Avoid cleaning with alcohol-based products. It can affect the transparency of the visor.

Sharp products such as knives, razor blades, etc. can scratch the visor and subsequently affect your vision.

Follow the attached assembly instructions when mounting the visor. When the protective plastic is removed, the visor should be mounted directly onto the holder to prevent scratches/damage to the surface.

Avoid hard shocks and knocks to the visor.

The visor is intended to be used at room temperature.

The product can be used several times.

Restrictions on use

The visor does not protect against steam that can get behind the visor.

The visor does not purify inhaled air.

Repeated use of disinfectants can give the visor a dull surface.

The visor is not intended to be used for protection against shocks/knocks.

The product is not intended to be sterilised.

Durability

The product can be used repeatedly until the visor has a dull or scratched surface. Then the product should be discarded. The product should not be stored in direct sunlight or in excessively hot conditions.

Under normal conditions of use, the product can be used for just over a year.

Repackaging/distribution

The product is shipped in a 60-pack in the existing customised packaging. If the product is to be forwarded, it is recommended to use the existing packaging or to repackage it in shock-absorbing packaging.

This manual should be made available to each user.

The assembly instructions should also be made available to each user.

Labeling

The product is labelled according to MET5659.



Ide-Pro = manufacturer

1 = product's optical class

MET5659 (covid-19) = Current standard/method

3 = protection against liquid and splashes

Product requirements

The product meets the requirements of PPER (EU) 2016/425.

Tests

The product has been tested according to MET5659.

Product checking

The EU type check is carried out by RISE Research Institutes of Sweden B, Box 857, SE-501 15 Borås.

Notified Body no. 0402

Harmonised standards

MET5659 edition 1

EU declaration

The EU statement of conformity can be downloaded at www.ide-pro.com

This manual should be available to each user.